

Instruction to the New Alignment Tool A-800


Step 1:

Put the starter rod (6 inches long) across the negative mold at the middle of lateral and medial malleoli then pour plaster.

Remove the starter rod to have a through hole in the positive model.

Step 2:


Remove the nut on the alignment tool by hand

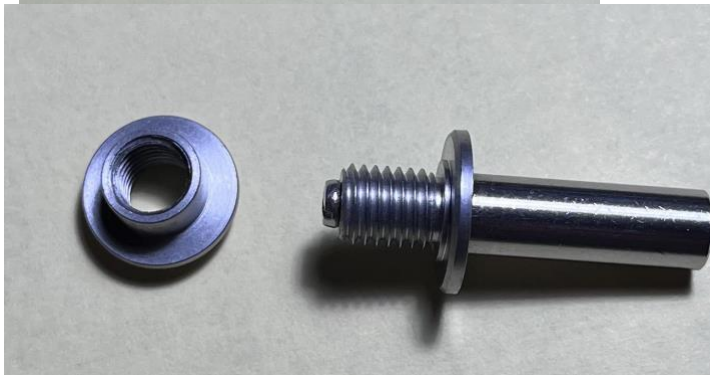

Step 3:

Disassemble the 810 hinges, put the pins and clips aside for later use.

Stack the rotating pieces onto the nut.

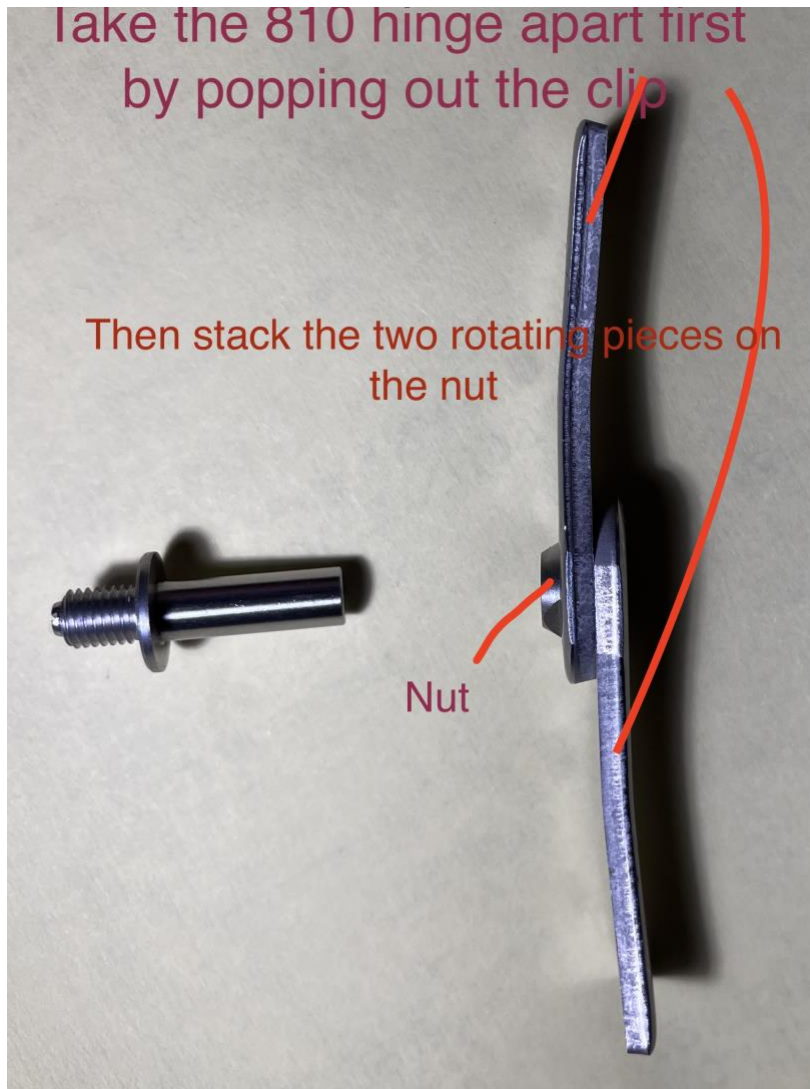

Step 4:

Assemble the shorter rod – thread in by hand



Step 5: Insert the assembly into the hole on the plaster

Step 6: Use nails or adhesives to secure the hinge on the plaster

Step 7: Do the same for the other side, then start pulling plastic